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**Instructions for use****Product description**

Coverflex fast ist ein non-sterile permanently elastic tubular bandage for use on human skin. It is single-use.

**Indications**

- Coverflex fast can be used  
by lay persons and healthcare professionals
- as a moist dressing for application in combination with primary dressings for the treatment of acute and chronic wounds;

**under supervision of healthcare professionals**

- as a moist dressing in dermatology when moistened for short-term (< 30 days) treatment in the acute phase of atopic eczema in a special moisture and ointment therapy for application in combination with liquids, ointments and creams;
- as a dry dressing in dermatology for long-term treatment in the chronic phase of atopische eczema for application in combination with extrema such as ointments, creams, powders, tinctures/lotions and pastes;
- as an aid in long-term prevention of skin irritation for application in combination with compression, zinc paste, support, cast, adhesive and cohesive bandages;

**Precautions**

- discontinue use and seek immediate medical advice, if skin condition worsens;
- Prior to use as moist or dry dressing in dermatology: seek advice from a healthcare professional.

- During application and removal of the tubular bandage it is essential to keep the primary dressing in place.

**Application**

- As secondary dressing to retain primary dressings
- Select right size (see table with declaration of circumference). Cut off required length of Coverflex fast (dependent on the size of the primary dressing and length of body part).
- Apply Coverflex fast over the end of the concerned body part.
- Apply Coverflex fast carefully upwards over the primary dressing.

In some cases an additional fixation of primary dressing might be necessary.

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**II. As a moist dressing when moistened (for the acute phase of atopic eczema as a special moisture and ointment therapy)**

- Select right size (see table with declaration of circumference). Cut off required length of Coverflex fast (dependent on the length of the body part).
- Apply the chosen extema (e.g. ointment) to the skin.
- Immerse the bandage in lukewarm water and wring it out.
- Apply Coverflex fast carefully upwards over the affected area of the body.

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**III. As a dry dressing during chronic phase of atopic eczema for application in combination with extema such as ointments, creams, powder, tincture/lotions and pastes**

- Select right size (see table with declaration of circumference). Cut off required length of Coverflex fast (dependent on the length of the body part).
- Apply the chosen extema (e.g. ointment) to the skin.
- Apply Coverflex fast carefully upwards over the primary dressing.

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- select right size (see table with declaration of circumference). Cut off required length of Coverflex fast (dependent on the size of the primary dressing and length of body part).
- Apply Coverflex fast over the end of the concerned body part.
- Apply Coverflex fast carefully upwards over the primary dressing.

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**II. As secondary dressing to retain primary dressings**

- Select right size (see table with declaration of circumference). Cut off required length of Coverflex fast (dependent on the size of the primary dressing and length of body part).
- Apply the chosen extema (e.g. ointment) to the skin.
- Immerse the bandage in lukewarm water and wring it out.
- Apply Coverflex fast carefully upwards over the affected area of the body.

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**III. As a moist dressing when moistened (for the acute phase of atopic eczema as a special moisture and ointment therapy)**

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