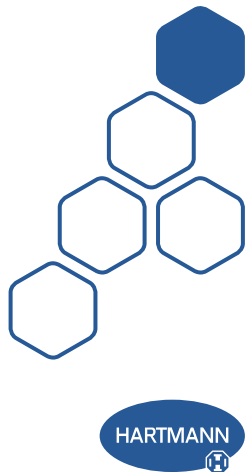
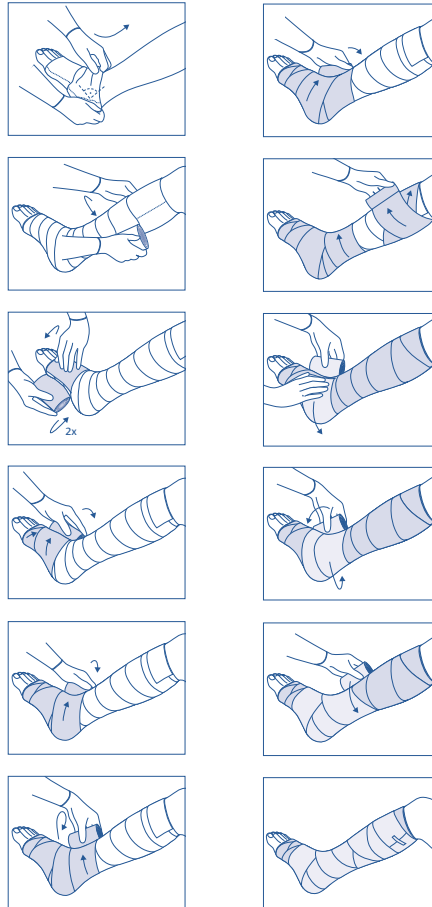
Pütter®binde

Kräftige Kurzzugbinden
Strong short-stretch bandages
Bandes résistantes à allongement court
Sterke zwachtels met korte rek
Bende a corta trazione forte
Vendas resistentes de corta elasticidad
Ligaduras resistentes de curta tração
Ανεκτικοί επίδεσμοί μικρής τάσης
Mocne opaski o niskiej rozciągliwości
Erős, rövid megnyúlású pólyák
Прочные бинты короткой растяжимости
Bandaje rezistente cu elasticitate redusă
Starka kortsträcksbindor
強力短拉力繃帶
أربطة متينة قصيرة المدى

1

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EN Instructions for use

General description of the device

Pütterbinde* is a brown-coloured, especially strong short-stretch bandage with textile elasticity and an extensibility of approximately 85 % for very high compression.

Intended purpose

Pütterbinde can be used for long-term compression therapy:

- treatment of chronic venous insufficiency (CVI) with and without venous leg ulcers (VLU)
- thrombosis/thrombophlebitis
- chronic oedemata of other origins (ankle brachial pressure index – ABPI 0.8–1.3)
- lymph oedemata

The product can be used on humans by healthcare professionals in clinical and home environment in combination with padding bandages and inner lining material. Pütterbinde is reusable.

Properties

Extensibility: approx. 85%.
Washable: Wash the bandage in a regular wash cycle at a temperature of 95°C.

Indication

The product can be used for the treatment of varicose veins, thrombosis/thrombophlebitis, chronic venous insufficiency (CVI), primary and secondary prevention of leg ulcer venous/venous insufficiency and oedemata of other origins (ankle brachial pressure index - ABPI 0.8 - 1.3).

Intended users

Pütterbinde is designed to be used by healthcare professionals who have received appropriate training. Pütterbinde can be used for the treatment of: varicose veins, thrombosis/thrombophlebitis, post-thrombotic syndrome, thromboprophylaxis, chronic venous insufficiency (CVI), venous insufficiency, angiodysplasia and other oedemata.

Composition

100 % cotton

Contraindications

Pütterbinde must not be used in case of:

- advanced peripheral arterial occlusive disease (PAOD)
- decompensated heart failure (NYHA II +IV)
- phlegmasia coerulea dolens

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DE – PAUL HARTMANN AG · Paul-Hartmann-Straße 12 · 89522 HEIDENHEIM, GERMANY
CH – IVF HARTMANN AG · 8212 Neuhausen